PHILIPS ZOOM!

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EN

PHILIPS ZOOM QUICKPRO TAKE-HOME WHITENING VARNISH 4-DAY TREATMENT KIT

To be only sold to dental practitioners.

First application is to be done by the client under the supervision of the dental professional. All other applications are done by the client in the convenience of their own home.

PRODUCT DESCRIPTION:

Philips Zoom QuickPro Take-Home Whitening Varnish kit is a 6% hydrogen peroxide (on the tooth) whitening varnish used to whiten discolored natural teeth. The QuickPro take-home kit is a 4-day kit with 8 applications (2 applications per day), applied by the client in their home. After application and a 30-minute wear time, client removes the whitening varnish by gently brushing or wiping it off.

IMPORTANT:

READ ALL INSTRUCTIONS WITH THE CLIENT PRIOR TO THE CLIENT APPLYING THIS PRODUCT

WARNINGS

- This product contains hydrogen peroxide.
- Avoid contact of the product with eyes as irritation or serious damage to eyes may occur. Rinse with water immediately if this occurs. If it comes in contact with skin, it may cause slight irritation. Contact with clothing may result in discoloration and/or damage.
- Harmful when swallowed. If nausea or vomiting occurs, notify your dentist or medical professional right away.
- Avoid exposure of known hypersensitive teeth and teeth with large pulps and cavities to the whitening varnish.
- Keep out of reach of children.
- The Philips Zoom QuickPro Take-Home Whitening Varnish kit should not be used more than 4x per year.
- Flammable: Do not use near open ignition source.
- Do not freeze or expose the whitening varnish to extreme heat.

CONTRAINDICATIONS

Current research has not evaluated the possible effects of whitening procedures on all client types. Philips recommends the following whitening candidates consult their medical doctor before use:

- Pregnant or lactating mothers
- Those currently treated for a serious illness, e.g. immune compromised AIDS, etc.
- Not for use by clients with physical limitations preventing them from applying layer 1 in 60 seconds or less.
- Not for use on persons under 18 years of age

FIRST AID INSTRUCTIONS

- For advice, contact Poison Information Center or a doctor immediately.
- If swallowed do not induce vomiting.
- If varnish gets into eyes, hold eyelids apart and flush the eye continuously with running water for at least 15 minutes.
- If clothing, skin or hair contact occurs, remove contaminated clothing and flush skin or hair with running water.

POTENTIAL ADVERSE EFFECTS

Adverse effects that may be associated with the use of a whitening product containing hydrogen peroxide include, but are not limited to:

- Sensitivity: Clients more susceptible to sensitivity are those with exposed root surfaces, defective restorations, exposed dentin, oral tissue injury, periodontal disease, recession, untreated caries, cracked teeth, abfractions, open cavities or leaking fillings.

KIT CONTENTS

- Cheek retractor
- 8 Whitening varnish vials (labeled: V1...V8 and with White Caps)
- 1 Sealant vial (labeled S1 and with Black Cap)
- 8 white brushes
- 8 black brushes
- Instructions for Use
- Shade Guide

DENTAL PROFESSIONAL INSTRUCTIONS

CASE SELECTION

- Only patients that are capable of applying layer 1 within 60 seconds should be selected for treatment with this product.
- Discoloration of teeth can be caused by: staining from food, drinks and tobacco with penetration into the tooth enamel/dentin, age dependent degenerative changes, tetracycline or minocycline medications, fluorosis, endodontic treatment and genetically discolored teeth.
- Review client's current dental health and medical history to ensure that client is in good oral and physical health. Clients with any medical conditions should consult with their medical doctor before beginning any whitening treatment.
- Clients with composite restorations must be advised that existing composites may require replacement, since only natural teeth will whiten.
- For clients requiring aesthetic restorations, wait 2 weeks after the last whitening treatment to allow for color stabilization. Do not place adhesive restorations on teeth within 7-10 days of any tooth whitening treatment as recent studies show transient decrease in bond strength during that period.
- Methyl methacrylate provisional or temporaries may discolor and redden during the whitening process. However, composite or polycarbonate materials will remain unaffected.

CLIENT EXPECTATIONS

Set realistic expectations about potential results based on oral exam and explain results vary. Clients can expect a noticeably whiter smile when the Philips Zoom QuickPro Take-Home Whitening Varnish kit protocol is followed per the instructions provided. Anatomy, calcifications, age, tooth condition, hygiene, etc., determine the results each client will attain. Teeth with yellow and brown stains will achieve greatest success while those with gray shading from tetracycline or other medications may experience little to no results. For gray shades or very dark stains we recommend a combination of treatments that may include the Philips Zoom Chairside procedure followed by take-home whitening.

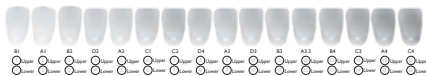
STORAGE INSTRUCTIONS

Refrigeration required: between 2°C and 8°C (36°F and 46°F). Storage temperatures above 8°C (46°F) will result in reduced shelf life. Remove kit from refrigeration 5 minutes prior to patient application. Inform patient that kit should be kept at home at room temperature.

FIRST TREATMENT: UNDER SUPERVISION OF THE DENTAL PROFESSIONAL

Before beginning the Philips Zoom QuickPro Take-Home Whitening Varnish treatment, perform a prophylaxis to ensure the client is free of decay, calculus and easily removable extrinsic stains. All restorations should be intact or well-sealed with adequate temporary material.

1. For best results provide client with the Philips Zoom QuickPro Take-Home Whitening Varnish kit immediately following a prophylaxis or light polishing.
2. Review the contents of the Philips Zoom QuickPro Take-Home Whitening Varnish kit and the Instructions for Use with the client.
3. Each kit includes a Shade Guide. The Shade Guide can be used to show the client their starting shade so they can better see how white they've gotten after completing their whitening treatment.

Shade Guide

For a healthy, beautiful smile.

Shade guide shown is for patient reference only. Actual shade assessment should be performed by a dental professional with Vita Classical Shade Guide (Vita Zahnfabrik, Germany) arranged in value order. © 2014 Koninklijke Philips N.V. (KPNV). All rights reserved. Philips and the Philips shield are registered trademarks of KPNV. Zoom is a trademark of Discus Dental, LLC.

**PHILIPS
ZOOM!**

Upper: mark the number that represents clients starting shade for upper teeth

Lower: mark the number that represents clients starting shade for lower teeth

For best results using the Philips Zoom QuickPro Take-Home Whitening Varnish treatment clients should have a starting shade of A2 or darker.

4. With the client in front of a mirror, watch and give direction as needed, as the client applies the first treatment (since client will have just had a polishing it is not necessary for them to brush and floss their teeth for this first treatment). Once the treatment is applied and both the dental professional and the client are comfortable with the instructions, client leaves the office with the varnish on and will remove it 30 minutes later by either brushing or wiping off.
5. All other Philips Zoom QuickPro Take-Home Whitening Varnish treatments will be done by the client at home.

PRECAUTIONS

- Wait 2 weeks after whitening to replace bonded restorations.
- Advise clients that if irritations such as redness, swelling or soreness of the gums or the mouth occurs, discontinue use and consult their dentist.
- Advise clients to not eat, drink or smoke while varnish is on the teeth.

CLIENT INSTRUCTIONS

STEP BY STEP INSTRUCTIONS:

- a) Familiarize yourself with the WARNINGS presented on page 10 before proceeding.
- b) Brush and floss your teeth.
- c) Insert grey cheek retractor. For easiest results, squeeze the two sides together and insert from left to right and release. The arrow should be pointing up and facing the mirror. The cheeks should be pulled back exposing teeth. Refer to image 1.
- d) Use tissues to thoroughly dry gums and teeth. Remember to swallow naturally during the application process. If excessive

saliva pools around the lower teeth, use a tissue or cotton swab to keep gums and teeth dry.

- e) Open vial 1 (hydrogen peroxide), dip in a white applicator brush. Beginning with the bottom teeth and starting from back to front, brush upwards away from the gum line with a smooth even stroke. Refer to image 2. **Layer 1 must be applied within 60 seconds.** Try not to go over the same area repeatedly. If the first stroke did not cover the entire tooth, turn the brush and apply on the remaining section of the tooth. Re-dip the brush and repeat with the next tooth. A thin layer of varnish is all that is required and will dry in about 30 seconds. Applying too much will result in longer drying time. Varnish is dry when it is no longer sticky to the touch. Unused portion of vial 1 and the applicator brush should be disposed (refer to Disposal Instructions).
- f) Open vial 2 (sealant), dip in a black applicator brush and quickly apply beginning with the bottom teeth, moving back to front and using the same technique as with layer one. Unused portion of vial 2 may be re-capped and placed back in the kit. The applicator brush should be disposed.
- g) Once the sealant is dry (takes about 30 seconds) remove the cheek retractor. Rinse off with warm water, dry and return to the QuickPro kit.
- h) Do not rinse, eat or drink for 30 minutes while the varnish is on.
- i) After 30 minutes, remove the whitening varnish by using your toothbrush or wiping off with a paper towel or tissue.
- j) We recommend applying QuickPro Take-Home Whitening Varnish 2x per day for 4 days for maximum results.

Remember: Keep the whitening varnish on your teeth for 30 minutes and do not eat or drink during this time. While little sensitivity has been reported with the use of QuickPro Take-Home Whitening Varnish, every person is unique. If you experience sensitivity, a mild over-the-counter pain medication can be taken after the whitening varnish has been removed.

POST-TREATMENT

Following each QuickPro take-home whitening session, you should refrain from eating or drinking any dark colored food or beverages for 30 minutes. To continue good oral hygiene, we recommend you use a Philips Sonicare power toothbrush to help prolong the benefits of your whitening treatments.

STORAGE INSTRUCTIONS

Store between 15°C and 25°C (59°F and 77°F) in a dark, dry place. Clients: do not refrigerate. Storage temperatures above 25°C (77°F) will result in reduced shelf life.

DISPOSAL INSTRUCTIONS

Unused portions of product should be left open and allowed to dry prior to disposal. Allow the applicator brush to dry prior to disposal.

WARRANTY

Philips warrants that reasonable care has been used in the design and manufacture of this product. This warranty is in lieu of and excludes all other warranties not expressly set forth herein, whether express or implied by operation of law or otherwise, including, but not limited to, any implied warranties of merchantability or fitness for a particular purpose.

If you have any questions about the use of this product or concerns during the use of this product contact your dental professional.

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© 2014 Philips North America, Inc. (KPNVA). All rights reserved.